

Protocol Name: **LARGE BODY LUMBAR**

Body Part: **LUMBAR**

Coil: **HEAD, NECK AND SPINE ARRAY**

Sequences	Seq Name	Image Plane	# Slices	TR	TE	Slice Thick	FOV	NSA	Matrix	Phase Dir
2 Plane Loc	gre	2 plane	6	7.8	3.69	6	300	1	256	R>L
Sag T2	tse	Sagittal	17	3150	92	3	260	2	320	H>F
Sag T1	tse	Sagittal	17	511	9.4	3	260	2	256	H>F
Sag STIR	tse	Sagittal	17	3080	52	3	260	3	256	H>F
Axial T1	tse	Axial	34/16	4410	106	3.5	180	1	320	A>P
Axial T2	tse	Axial	34/16	750	9.8	3.5	180	2	320	A>P

Details: Be sure to use the B1 filter option to minimize b1 artifact